

ReSound Unite™ Remote Control 2



Product Description

The ReSound Unite™ Remote Control 2 is for clients who appreciate an easy overview of their hearing instrument and accessory settings. They can use it to change programs, adjust the volume, and control streaming from the TV Streamer 2 and Mini Microphone.

The Remote Control 2 allows the client to adjust the volume both monaurally and binaurally. When in streaming mode, they can individually adjust volume of the hearing instrument vs. streamed audio volume.

The Remote Control 2 has a high contrast LCD display so clients can clearly see their instrument settings at a glance.

With the simple keypad, it's easy to choose programs, control volume, and mute hearing instruments. The elegant look and small, handy feel of the Remote Control 2 makes it a pleasure to stay in control.

Easy to connect

- To pair: just press the pairing button on the Remote Control 2 and open and close the battery door of the hearing instruments within 20 seconds
- Pairing can also be done through the ReSound Aventa® 3 fitting software

Easy to use

- At the push of a button you can control volume, mute/unmute, and change programs
- Volume can be adjusted monaurally as well as binaurally
- 'Home' button lets you return to your default settings
- The 'Streaming' button activates streaming from your TV Streamer 2 or Mini Microphone (optional streaming devices)
- Rechargeable with battery status indicator
- Mechanical ON/OFF switch to avoid unintentional activation

2.4 GHz wireless Technology

- A smart, robust, and reliable standard that has a strong, clear, and stable signal

ReSound North America
8001 Bloomington Freeway
Bloomington, MN 55420
1-888-735-4327
consumerhelp@gnresound.com

ReSound Government Services
8001 Bloomington Freeway
Bloomington, MN 55420
1-800-392-9932
VAinfo@gnresound.com

ReSound Canada
303 Supertest Road
Toronto, Ontario M3J 2M4
1-888-737-6863

ReSound
rediscover hearing

Description

- 1

ON/OFF switch
- 2

Display
- 3

Streaming (optional TV Streamer or Mini Microphone)
- 4

Home
- 5

Hearing instrument microphone mute
- 6

Increase volume
- 7


Decrease volume
- 8

Select left hearing instrument only
- 9

Select right hearing instrument only
- 10

Program change
- 11

Pairing

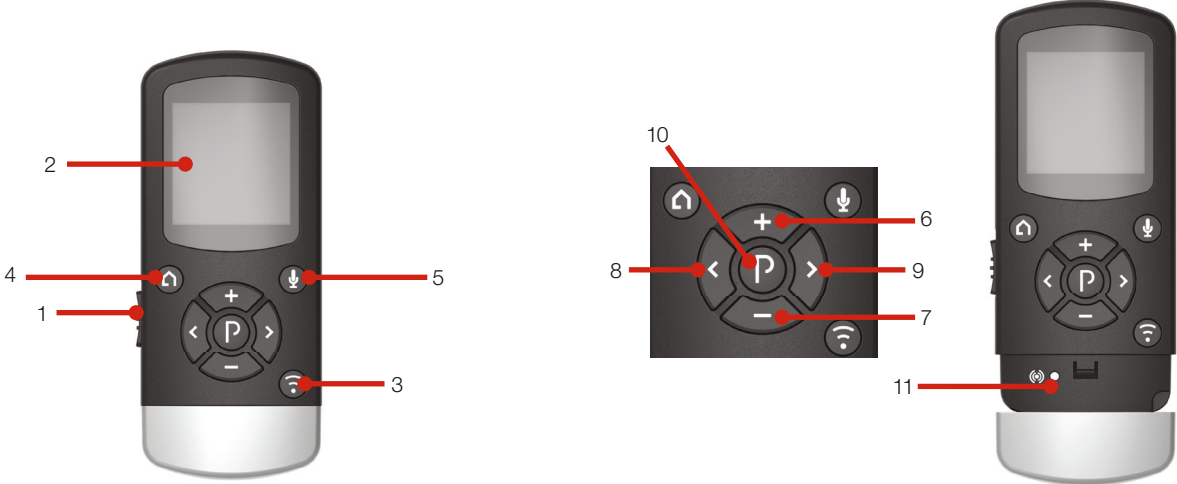

Technical data

Dimensions	98 x 42 x 15 mm
Weight	35 g
Display	128 x 128 pixel colour display
Power supply	Build-in rechargeable battery
Battery charge time	Max 2.5 hours using the charger supplied with the unit
Wireless connections	2 hearing instruments per Remote Control 2
Wireless frequency	2.4 GHz
Wireless range	Up to 3 meters
Communication speed	2 Mbit/s
Emission	Max. +4 dBm In compliance with all relevant standards
ESD tolerance	±4 kV contact ±8 kV air
Operating temperature	32 to 131 °F
Storage temperature	-4 to +140 °F