

Cochlear™ Nucleus® Kanso™ and ReSound

# Wireless Pairing & Streaming

Maximize your hearing with both ears!



## Microphones

Note: When bimodal pairing, first ensure hearing devices are programmed to opposite ears.

### 1 Turn off ReSound Hearing Aid and Kanso Sound Processor



Press and hold Kanso button until the light is a steady orange.

### 2 Turn on microphone

Note: Ensure microphone is fully charged.



Switch on.

### 3 Locate and press the round pairing button on back by the clip



20 seconds

Note: The LED will flash yellow while device is in pairing mode for 20 seconds.

### 4 Turn on ReSound Hearing Aid, pause 2 seconds, then turn on Kanso Sound Processor



Press the Kanso button to turn on.

## TV Streamers

### 1 Plug power and audio cables into TV Streamer and connect to audio source



### 2 Turn off ReSound Hearing Aid and Kanso Sound Processor



Press and hold Kanso button until the light is a steady orange.

### 3 Press pairing button



20 seconds

Note: The LED will flash yellow while device is in pairing mode for 20 seconds.

### 4 Turn on ReSound Hearing Aid, pause 2 seconds, then turn on Kanso Sound Processor



Press the Kanso button to turn on.

## Microphones and TV Streamers

### 1 Turn on ReSound Hearing Aid then Kanso Sound Processor



Press the Kanso button to turn on.

### 2 Turn on microphone or turn on TV Streaming to one or both hearing devices

Note: Streaming must be started and stopped on each device individually.

#### OPTION A

##### Kanso Sound Processor

Press and hold button for 2 seconds then release to stream audio.

#### OPTION B

##### CR210 Remote Control

Long-press the Telecoil button. Audio will start streaming through the microphone.

#### OPTION C

##### Using CR230 Remote Assistant

Long-press the Telecoil button.

#### OPTION A

##### Press and hold the push button on your ReSound Hearing Aid for 3 seconds



#### OPTION B

##### Using ReSound Unite Remote

If you have a ReSound Unite™ Remote Control (optional), simply press the streaming button on this Remote Control.



Note: Press and release again if you need to cycle to the next audio source. This is necessary if your audiologist enabled the Telecoil as this is the first streaming audio source.

# Wireless Pairing & Streaming

Maximize your hearing with both ears!



## Phone Clip – Pairing with Hearing Device

**1** Turn off ReSound Hearing Aid and Kanso Sound Processor



Press and hold Kanso button until the light is a steady orange.

**2** Turn on Phone Clip and remove silver cap



Switch on

Remove cap

**3** Press white pairing button



**20** seconds

Note: The LED will flash yellow white device is in pairing mode for 20 seconds.

**4** Turn on ReSound Hearing Aid, pause 2 seconds, then turn on Kanso Sound Processor



Press the Kanso button to turn on.

## Phone Clip – Pairing with Mobile Phone

Note: The Phone Clip must be paired to a Bluetooth source before it can be used.

**1** Turn on Phone Clip and remove silver cap



Switch on

Remove cap

**2** Turn on mobile phone's Bluetooth function

Refer to your mobile phone user guide for instructions.

**2** minutes

Note: During pairing mode the LED will turn blue.

**3** Press blue Bluetooth pairing button



**4** On your mobile phone search for new Bluetooth devices and select "Hearing Aid Device" from the list

Note: If your mobile phone asks for a pass code, enter "0000" (four zeros)

Your Phone Clip should now be paired with your mobile phone.

Note: If your mobile phone asks which Bluetooth service you wish to enable, select "Headset".

## Phone Clip – Managing Calls

**Incoming calls:**

To answer a call, press once.

To decline a call, double-click.

**During a call:**

To transfer a call started on your phone to the phone clip, press once

To put a call on hold, press and hold for 2 seconds.

**To make a call:**

To redial the last number, double-click.

To activate voice dialling, press and hold for 2 seconds.

## Phone Clip – Streaming Music

Connect the device to the Phone Clip using Bluetooth.

Start playing music on the device.

You hear the audio streaming from the device.

To stop streaming, stop playing music on the device.

Tip: Use the volume controls on the Phone Clip to change the volume on your hearing aid.



© Cochlear Limited 2016. All rights reserved. Hear now. And always and other trademarks and registered trademarks are the property of Cochlear Limited or Cochlear Bone Anchored Solutions AB. The names of actual companies and products mentioned herein may be the trademarks of their respective owners.  
© 2016 GN ReSound A/S. All rights reserved.  
ReSound is a trademark of GN ReSound A/S.

1 Wolfe, Jacq. "Why Two Are Better Than One." Presented at ACIA 2014, Nashville, TN.



MK604588 Rev C 2016.09