
Smart 3D™ app: Sound Enhancer

What is Sound Enhancer?

Sound Enhancer allows the user to fine-tune multiple settings in the ReSound Smart 3D app to personalize their hearing.

Why Sound Enhancer?

To empower patients to take control of their own hearing experience through solutions

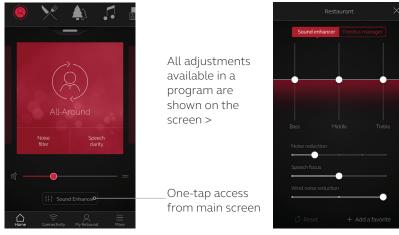

Sound Enhancer view


Entry to Sound Enhancer

to their biggest pain point: moving between and optimizing settings in different, challenging listening environments. Users can fine-tune bass, middle and treble frequencies, noise reduction, speech focus and wind noise reduction.

Using Sound Enhancer:

- Hearing aids must be in same program to use Sound Enhancer.
- Changes made to Sound Enhancer features are active in a program until reboot (unless saved as a Favorite).
- All adjustments available in a program are shown on the screen per program.
- Available in all technology levels, but Noise/Speech/Wind controls are only available in ReSound LiNX 3D™ 9.
 - o Only available in manual programs (i.e. not in streaming programs).
- Sound Enhancer is not available while the Acceptance Manager is active. When the Acceptance Manager period expires, Sound Enhancer automatically becomes available.

Sound Enhancer adjustments

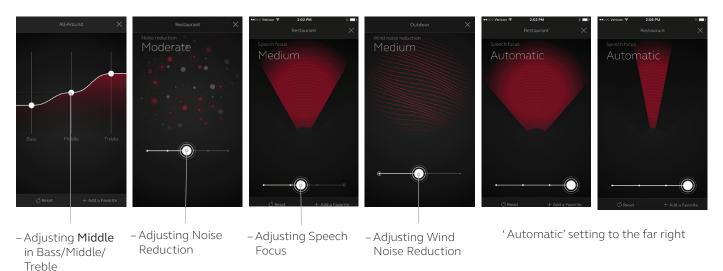

Smart 3D™ app: Sound Enhancer detail

Program	Noise	Speech	Wind
All Around	0	0	0
Restaurant	•	•	0
Outdoor	•	0	•
Music	•	0	0
Acoustic Phone	0		0
T-coil phone/loop (in normal program)	•		
DAI (in normal program)	•		
All streaming programs	•		

• Available with default settings

o Available in manual programs per feature if:

Noise: Noise Tracker II not set to Per Environment

Speech: Directionality set to adaptive (Autoscope or SoftSwitching)

Wind: Wind Guard enabled

Feature values for left and right ears have to be the same. For Speech this includes both Beamwidth and Directional Mix values.

Bass/middle/treble frequency ranges: 125Hz-1kHz // 1kHz-3kHz // 3kHz-7kHz

Manufacturer according to FDA:

ReSound North America

8001 Bloomington Freeway Bloomington, MN 55420 1-800-248-4327 resoundpro.com ReSound Government Services

8001 Bloomington Freeway Bloomington, MN 55420 1-800-392-9932 resound.com/governmentservices

Manufacturer according to Health Canada:

ReSound Canada

303 Supertest Road Toronto, Ontario M3J 2M4 1-888-737-6863 resoundpro.com



