

Tech Tips



ReSound GN

ReSound Smart Fit | Removal of Party & Traffic programs

To make software more intuitive and user-friendly, ReSound conducted extensive usability studies. We studied how people actually use fitting software and incorporated this information into the new design. We wanted to design to the 80/20 rule, which in the world of software generally means that 80% of users will only use 20% of the features. Therefore, we sought to identify the most common 20% of features and make those most accessible to the user, while making the other 80% less prominent but still available for advanced users.

In line with that philosophy, the party and traffic programs were removed from Smart Fit due to low usage and similar properties to other programs that were more commonly used. If you wish to replicate the party or traffic programs, you can do so using the information below.

To replicate a Traffic program in Smart Fit, start with an Outdoor program:

- Increase NoiseTracker II to “Considerable”
- Turn WindGuard off
- Adjust the G50 gain as shown in the chart below

Program	Input	125	250	500	750	1000	1500	2000	3000	4000	6000	8000
Traffic	G50		-6	-4		-3		-3		-3	-3	

To replicate a Party program in Smart Fit, begin with the Restaurant program:

- Increase NoiseTracker II to “Considerable”
- Adjust the G50 gain as shown in the chart below

Program	Input	125	250	500	750	1000	1500	2000	3000	4000	6000	8000
Party	G50		-6	-6				-3			-3	

Manufacturer according to FDA:

ReSound North America
8001 E. Bloomington Freeway
Bloomington, MN 55420
800.248.4327
resoundpro.com

Manufacturer according to Health Canada:

ReSound Canada
303 Supertest Road
Toronto, Ontario M3J 2M4
1.888.737.6863
resoundpro.com