


Made for Apple pairing

Bimodal use with a compatible hearing aid*

For the first time ever, you can stream sound from an Apple® device directly to both your Nucleus® 7 Sound Processor and compatible ReSound hearing aid**, simultaneously. So you can enjoy music, phone calls and entertainment from your Apple device, in both ears.

Important: Your hearing devices must already be linked together by your clinician before you can pair them both to your Apple device.

Pairing your sound processor and hearing aid to your compatible Apple device

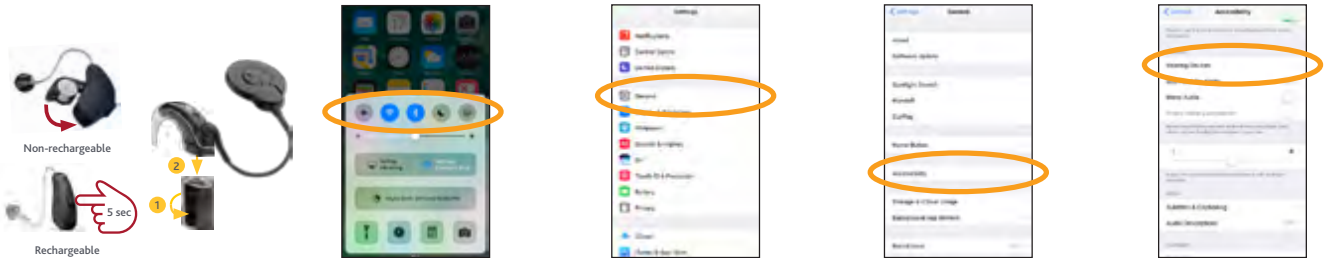

- 1 Turn off ReSound hearing aid*** and Cochlear Nucleus 7 Sound Processor.
- 2 To pair your sound processor and hearing aid with your compatible Apple device, ensure Bluetooth® is enabled. Then, open the 'Settings' menu.
- 3 Select 'General'.
- 4 Select 'Accessibility'.
- 5 Scroll down and select 'MFH Hearing Devices'.

Note: Before bimodal pairing, ensure the hearing aid on the non-implanted side is correctly assigned to that ear via the hearing aid fitting software.



- 6 Next, turn your sound processor and hearing aid on.***
- 7 Select your name when it displays on the screen.
- 8 Select Pair to confirm pairing. You will see this message twice.
- 9 When pairing starts:
 - blue flash for 4 seconds
 - 6 beeps.
 Pairing may take up to 120 seconds. When pairing is finished:
 - blue flash for 4 seconds
 - ripple tone.
- 10 Your sound processor and hearing aid will both be listed under 'your hearing device name' and you should now be able to stream sound directly to both.

Note: Don't start to stream audio until pairing is finished.

Pairing to more than one iPhone®, iPad® or iPod® touch

You can pair your sound processor and hearing aid with up to five compatible Apple devices, but you can only control them from one Apple device at a time. To manually change connection between devices you have previously paired to, turn off Bluetooth on the connected device, and turn on Bluetooth on the device you want to connect to. Your hearing devices will then automatically connect to the second device.

* For a list of compatible ReSound Hearing Aids visit www.cochlear.com/nucleus/compatibility

** For rechargeable ReSound LiNX Quattro™: press the push button for 5 seconds.

*** For more information refer to the Apple Accessibility Quick Guide.

Cochlear™ True Wireless™ device pairing

Cochlear Wireless Mini Microphone



- Turn off ReSound hearing aid^{***} and Cochlear Nucleus 7 Sound Processor.
- Turn on Cochlear Wireless Mini Microphone 2/2+.
Note: Ensure Wireless Mini Microphone is charged.
- Press pairing button.[^]
Press the pairing button once for channel 1, twice for channel 2, three times for channel 3.
Note: The LED will blink orange for 20 seconds while device is in pairing mode.
- Next, turn your sound processor and hearing aid on.^{***}
Note: The LED will flash blue to show pairing was successful.

Cochlear Wireless TV Streamer



- Plug power and audio cables into Wireless TV Streamer and connect to audio source.
- Turn off ReSound hearing aid^{***} and Nucleus 7 Sound Processor.
- Press pairing button.
Note: The LED will blink orange for 20 seconds while device is in pairing mode.
- Next, turn your sound processor and hearing aid on.^{***}
Note: The LED will flash blue to show pairing was successful.

Cochlear True Wireless device streaming

Cochlear Wireless Mini Microphone and Wireless TV Streamer



- Turn on ReSound hearing aid^{***} then Nucleus 7 Sound Processor.
- Turn on Cochlear Wireless Mini Microphone 2/2+ or Wireless TV Streamer.
- Streaming to one or both hearing devices
Note: Streaming must be started and stopped on each device individually.
- Streaming to both ears
Through Apple's accessibility menu you can control streaming to both the sound processor and hearing aid. Triple click the home button to access controls for turning on a streaming source and to adjust volume.^{***}

3a



Nucleus 7 Sound Processor
Press and hold the button on your sound processor for two seconds, then release the button.




Cochlear CR310 Remote Control
Press and hold the Telecoil button for two seconds, then release.

3b



Rechargeable ReSound Hearing Aid
Press and hold the push button on your ReSound Hearing Aid.

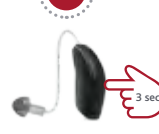

Non-rechargeable ReSound Hearing Aid
Press and hold the push button on your ReSound Hearing Aid.



ReSound Remote Control 2 (optional)
Press the streaming button on this remote control.